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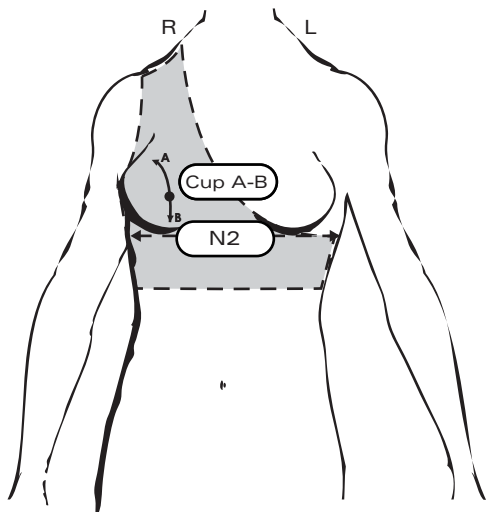
**Why, when and how to wear the Caroskin™ Radiotherapy Vest?****Intended indication:**

The unilateral bra serves to keep the contralateral breast away from the radiation field, while allowing free hanging from the ipsilateral breast.

**Goal audience:**

The patient group is defined as women who should receive radiation in the prone position on their chest after experiencing breast cancer.

**Part of the body for which the product is used:**

Both the left and the right chest.

**Practical tips**

- Skin must always be correctly hygienic and dry.
- Remove rings, watches, bracelets and / or other jewelry when putting on the Caroskin™ Radiotherapy Vest.
- Cut long nails short.
- Take your time to put on the Vest correctly.
- Do not cut loose hanging threads.
- For possible repairs, contact your prescriber.

**Wash & Care Advice**

- Washing at 30 °C of the Caroskin™ Radiotherapy Vest with Ph neutral soap extends the lifetime. Through the washing the clothing removes perspiration, skin fats and other substances that have influence on the elastic qualities of the vest.
- Frequent rinsing
- Do not wring, do not dry at heat source: allow air to dry

**Warning**

- Correct measurement:
  - Responsible: breast consultants
- Apply BH correctly:
  - Responsible breast consultants and nurses
- Maintenance of the bra:
  - see wash & care advice. This bra is for individual use and is standard or custom made. The use is only max 15 minutes per irradiation session, for maximum 15 days.

**How do you determine the bra size?**

The bra size consists of a number and a letter. The number (75-110) stands for the chest circumference (size or band size). The letter (A-E) stands for the cup size.

**Chest circumference (size or band size = N2 size)**

The chest circumference or band size is measured just below the bust. Take a tape measure and place the meter horizontally (front and back at the same height) around the chest just below the breasts. Exhale for a moment and let the meter fit tightly around your body, without pinching. The number of centimeters determines your circumference, rounded to 5 cm.

Chest size	75/80	85/90	95/100	105/110
<b>Measurement N2</b>	73 - 82 cm	83 - 92 cm	93 - 102 cm	103 - 112 cm

With the smallest breast circumference the hooks are closed on the 2nd row of eyes, with the largest breast circumference on the 1st row and this on both sides.

**The Cup size**

Important! this size is measured with a Bra on!

The cup size is determined by measuring the length of the breast (AB size). Take a measuring tape and place it at the start of the curve of the breast over the chest to the bottom of the breast border. The number of centimeters can be found in the table, which then correspond to a letter indicating the cup size.

Chest size	75/80	85/90	95/100	105/110
<b>A</b>	14 - 15 cm	15 - 16 cm	16 - 18 cm	-
<b>B</b>	15 - 17 cm	17 - 19 cm	18 - 21 cm	19 - 22 cm
<b>C</b>	17 - 19 cm	19 - 21 cm	21 - 23 cm	22 - 24 cm
<b>D</b>	19 - 21 cm	21 - 24 cm	23 - 25 cm	23 - 26 cm
<b>E</b>	24 +	25 +	26 +	-

\* If the measured centimeters can't be found in the tables, we opt for made to measure.

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