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Why and when to use the Caroskin™ Stoma Belt?**Intended indication:**

The Caroskin Stoma Belt provides optimal support for the stoma throughout the day and prevents bowel bulging when there is too much pressure on the abdominal wall due to intense physical activity. The Stoma Belt is also used for postoperative support of the abdominal wall.

Goal audience:

Stoma wearers:

- For optimal support of the stoma throughout the day.
- Prevention of prolapse formation (i.e. a bulging of the intestines due to too much pressure on the abdominal wall) during physical activities (sports, gardening...).
- Post-operative support of the abdominal wall.

Advantages

- High wearing comfort
- Good support
- High freedom of movement
- Optimal fit through the use of adjustable straps
- Easy to put on and take off
- There are 5 standard sizes available, but customization is also offered.
- Use on left and right is possible.
- Suitable for different ring* openings.
- Latex free
- Silicone fabric around stoma opening ensures a better grip and prevents the belt from shifting.

*The StomaGord has adapter rings in 6 inner diameters. Thus, the belt is compatible with almost any ostomy pouch. (Diameter rings: 40, 50, 60, 70, 80, 100 mm).

Practical tips

- Skin must always be correctly hygienic and dry.
- For possible repairs, contact your prescriber.

Wash & Care Advice

- Washing at 30 °C of the Caroskin™ Stoma Belt with Ph neutral soap extends the lifetime. Washing of the Stoma Belt removes perspiration, skin fats and other substances that have influence on the elastic qualities of the belt.
- Wash with even colors
- Frequent rinsing
- Do not wring, do not dry at heat source: allow air to dry

Warning

- Correctly apply and determine the size of the Stoma Belt:
Responsibility: doctor or nurse
- Maintenance of the belt:
See wash & care advice.

Sizes table

* It is also possible to have a Stoma Belt custom made for the patient.

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