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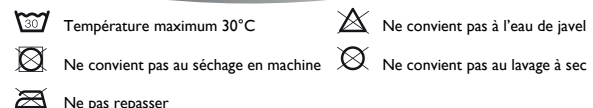
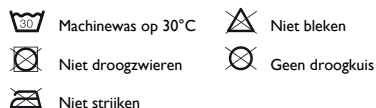
What is the Caroskin™ Lymph Care concept?

The Caroskin™ Lymph Care concept consists of different Caroskin™ products which have been put together to guarantee you the best possible lymphoedema compression treatment using top quality products.

These products are:

- Caroskin™ Gloves, Arm sleeves, Stockings, Compression Bra, ... :
 - Flat knitting **seamless**
 - S-Line Strong
 - S-Line Comfort
 - Flat knitting **with seam**
 - Comfort
 - Strong

Why, when, and how are Caroskin™ Comfort Lymph Care compression garments worn?

The standardised range of Caroskin™ Comfort Lymph Care compression garments is an extensive range of fully fledged compression class II compression garments which are designed to prevent or treat lymphoedema in the arms or hands, legs, ... by means of constant, sustainable, and therapeutic compression.

Caroskin™ Comfort Lymph Care compression garments have the following features:

- Made to measure or standard
- Therapeutic compression function (compression class II).
- A range which takes account of the different circumferences of the hands or arms.
- An open finger glove can replace a closed finger glove to give more comfort through free finger tips.
- Is anti-allergic – consists of natural polymers.
- Half knitting at the ends of the sleeve = not as thick: easy to combine with a glove.
- S-Line: soft and comfortable elasticity ensures plenty of comfort due to the seamless knitting technique.
- Comfort quality: Advantages of the natural crustacean extracts:
 - Lightly hydrating
 - Bacteriostatic features
 - High transpiration absorption

By wearing the compression garments in the prescribed manner and maintaining them in the correct manner, you will achieve the desired results.

Practical tips:

- Always keep your skin clean and dry.
- Remove rings, watches, bracelets, and/or jewels when putting on Caroskin™ Comfort Lymph Care compression garments.
- Cut any long nails short (to wear a closed glove).
- Take your time when putting the garments on.
- Contact the prescribing doctor if you think repairs may be needed.
- For warranty please keep the original packaging.

Washing and maintenance advice

- Washing Caroskin™ Comfort Lymph Care compression garments frequently (e.g. every 2 days) at 30°C using an ecological detergent extends their lifespan.
- Wash dark colours separately.
- Rinse frequently.
- Do not wring and do not dry near a heater, in an airing cupboard, or in direct sunlight: pat dry and air-dry.
- The Caroskin™ Pressure Garments retain their therapeutic pressure value for a minimum period of 2-3 months (if used and maintained correctly). The therapeutic pressure will last longer if you are able to switch between several identical pressure garments. However, re-measurement of the patient should be considered in the event of rapid increase or decrease of circumference.

Warning

- Protect your arms or hands against direct sunlight or rapid temperature changes (e.g. from Turkish baths, saunas, sunbeds etc.).
- Observe the rules of good hygiene.
- Only use pH neutral products for the maintenance of your Lymph Care compression garments. Otherwise the fibres may be affected and this can reduce the pressure.
- Since lymphoedema is a very complicated subject and each patient presents a separate case, the doctor who treats you will prescribe personalised treatment.

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