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EN

Caroskin™ Pressure Garments

Why, when and how should you wear the Caroskin™ Pressure Garments?

- The Caroskin™ Pressure Garments apply an even and constant pressure to the skin surface for the preventative treatment of hypertrophic and keloid scarring. To achieve positive results, we recommend wearing the Caroskin™ Pressure Garments daily for 20-24 hours per day during a period of 12 to 18 months.
- You will obtain the desired results by wearing the pressure garments in the described manner and maintaining them correctly.
- The Caroskin™ Pressure Garments are available in a unique range comprising 2 different types of pressure garment:

1). The Caroskin™ COMFORT Pressure Garments

are top-of-the-range pressure class II pressure garments developed especially to increase the patient's comfort significantly. The Caroskin™ COMFORT Pressure Garments contain natural extracts of shellfish, providing a therapeutic pressure function as well as a healing impact on micro wounds, anti-bacterial properties, hydration, and other benefits (cf. below). We recommend these pressure garments for initial therapeutic treatment. The Caroskin™ COMFORT Pressure Garments have greater elasticity, guaranteeing continued correct circular pressure even if the patient gains weight. However, re-measurement should be considered in the event of extreme weight gain.

SUMMARY: Caroskin™ COMFORT Pressure Garments

- Is a natural product;
- Biologically decomposable
- Haemostatic (micro wounds)
- Lightly hydrating
- High absorption of perspiration
- Increased comfort due to higher elasticity than polyamide pressure garments
- Reduces bacterial growth
- Anti-allergic – consists of natural polymers

2). The Caroskin™ STRONG Pressure Garment

a therapeutic pressure suit with high pressure class II and lower elasticity compared to Caroskin™ COMFORT Pressure Garments, exerting pressure on persistent scarring.

When should you replace the Caroskin™ Pressure Garments?

Both types of Caroskin™ Pressure Garments retain their therapeutic pressure value for a minimum period of 2-3 months (if used and maintained correctly). The therapeutic pressure will last longer if you are able to switch between several identical pressure garments. However, re-measurement of the patient should be considered in the event of rapid weight gain or loss or rapid growth in children.

Practical tips

- Skin must always be kept hygienic and dry.
- Remove rings, watches, bracelets and other jewellery when putting on the Caroskin™ Pressure Garments.
- Keep your nails short.
- Take your time to put on the garment. If you are using Caroskin™ pressure socks, turn the sock inside out except the heel, insert your foot in the sock and pull up the sock carefully.
- Correctly measured garments will feel relatively 'oppressive' at first. Your body will need a few days to get used to the exerted pressure. Pay attention to excessive pressure, which is expressed through rapid cooling off or a tingling sensation in the extremities. Contact your prescribing doctor.
- Do not cut off loose threads.
- Contact your prescribing doctor if the garment needs to be repaired.

Wash & Care Advice

- Regular washing of Caroskin™ Pressure Garments with Caroskin™ Cleaning Soap at 30°C extends their service life (e.g. every 2 days). Washing the pressure garments removes perspiration, skin oils and other substances that affect the elastic qualities of the garments.
- Caroskin™ Cleaning Soap
- Is an environmentally friendly laundry product developed especially to clean pressure and silicone products thoroughly.
- Reduces bacterial growth.
- Use: Machine wash: 8 tablespoons = 120 ml, Hand wash: 1 tablespoon = 15ml (depending on the size of the garment you are washing)
- Close any Velcro before washing.
- Wash dark colours separately.
- Rinse thoroughly.
- Do not wring, do not dry near a source of heat: air-dry.

Warning

- Damaged skin – scars must be protected from dangerous UVA-UVB radiation for a period of 5 years. Caroskin™ Pressure Garments provide 50 UPF protection.
- Only use the products in combination with the Caroskin™ Pressure Garments, since other products may affect the fibres,

Caroskin™ Silicone Garments

Why, when and how should you wear the Caroskin™ Silicone Garments?

- The Caroskin™ Silicone Garments provide the required semi-occlusivity for the skin surface to be treated. Semi-occlusivity has an impact on the colour, thickness and elasticity of the skin surface requiring treatment.
- To achieve positive results for preventive treatment, the silicone garments must be worn in combination with pressure garments. In this case, the silicone garments are worn at night and the pressure garments during the day.
- For existing scars that are no longer changing, the pressure garments are no longer effective. Only the silicone garments will have an effective impact. Wearing a silicone garment for 20 hours per day will have an impact on the redness (if the scar is still red), thickness and elasticity of the scar. Wearing a silicone garment may help immobilised skin surfaces regain suppleness. After a certain period (depending on the size, type and location of the scar), the affected area may regain mobility.
- For optimal effectiveness of the Caroskin™ Silicone Garment therapy, perfect hygiene of the skin and product is vital. Please read the washing instructions carefully.
- Caroskin™ Silicone Garments are supplied with Caroskin™ GLIDE Gel, which is used to make putting on the silicone garments easier and protect the silicone layer. Rub an adequate quantity of the gel on the cleaned skin and in the silicone product and glide the silicone garment over the skin area to be treated. The Caroskin™ GLIDE Gel also contains ingredients that reduce itching, are anti-bacterial and provide intense hydration. Avoid using other 'oily' substances since these deteriorate the silicone and may cause a rash.

When should you replace the Caroskin™ Silicone Garments?

- It is not necessary for the Caroskin™ Silicone Garments to exert pressure. They are intended to create a semi-occlusive climate (i.e. a sweaty climate). As long as the Caroskin™ Silicone Garments contain silicone, they continue to provide the required semi-occlusivity to continue the therapy. You will lengthen the service life of the garment if you are able to switch between several Caroskin™ Silicone Garments.
- Under normal circumstances and if the wash and care instructions are followed correctly, Caroskin™ Silicone Garments can be worn for 3 months.

Practical tips

- The skin and the Caroskin™ Silicone Garments must always be hygienic and dry before the garments are worn.
- Always use the associated products (Caroskin™ GLIDE Gel – Caroskin™ Cleaning Soap) to put on the garments and maintain them.
- Remove rings, watches, bracelets and other jewellery when putting on the Caroskin™ Silicone Garments.
- Take your time to put on the Caroskin™ Silicone Garments. Do not pull open the garment, but glide it over the body part in question. Learn to use the right quantity of Caroskin™ GLIDE Gel to put on the garments in a comfortable manner.
- To put on Caroskin™ silicone gloves, it is easiest to turn the glove inside out so that you can see the fingers. Put a bit of Caroskin™ GLIDE Gel in the tips of the fingers, rub the gel into the entire hand, slide your fingers into the glove and then pull the glove over the hand. Do not pull the knitted tricot part too hard.
- Do not cut off loose threads.
- Contact your prescribing doctor if the garment needs to be repaired.

Wash & Care Advice

- Use the Caroskin™ Cleaning Soap to wash the silicone garments.
- Do not use laundry softener, bleaching or pre-laundering agent.
- Do not tumble dry or dry near a source of heat or in the sun.
- Do not iron.
- Do not dry-clean.
- Close any Velcro before washing.
- Wash daily in the washing machine at 90°C (40°C if combined with elastic tricot) or wash in hot water with Caroskin™ Cleaning Soap. Rinse thoroughly in warm, running water. Machine wash: 8 tablespoons = 120 ml, Hand wash: 1 tablespoon = 15ml (depending on the size of the garment you are washing)
- When not using the Caroskin™ Silicone Garments, store them in a dry, cool place at a temperature below 25°C and protect from sunlight.

Warning

- The Caroskin™ Silicone Garments provide a semi-occlusive climate for your skin. This means that treatment must occur in increasing stages to let your skin get used to the semi-occlusive climate. We start with 2 hours per day, adding an hour per day up to approximately 20 hours per day if worn alone or 8 hours per day if worn in combination with pressure garments. Depending on the scar, results will start to show after about 3 to 6 months.
- A limited number of people have reported skin irritation. However, most skin irritation is due to inadequate hygiene-related preventative measures. In case of irritation, stop using the garment for 2 to 3 days before restarting treatment. Start anew as in the early stages and maintain strict hygiene of the skin and product.
- In the event of permanent contraindications: contact your doctor.
- We can only guarantee the correct functioning of the product if maintained correctly at all times with the correct cleaning agents (Caroskin™ Cleaning Soap) and used with the correct application agents (Caroskin™ GLIDE Gel).
- Do not apply directly to open wounds.
- In case of dermatological problems, the Caroskin™ silicone garments must be used in consultation with the doctor and therapists.
- Always use Caroskin™ GLIDE Gel to put on the Caroskin™ Silicone Garments!

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