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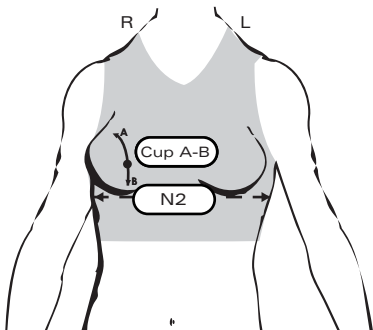
Why, when and how to wear the Caroskin™ Compression Bra?

Intended indication:

- To smoothly drain fluid in the breast, underarm and back.
- To prevent and/or reduce swelling and possibly to prevent scarring.
- To support breast and breast prosthesis after amputation.
- After surgery:
 - breast-conserving surgery;
 - breast reconstruction;
 - breast augmentation or breast reduction;
 - mastectomy
- Lymphedema under the armpit.
- Acute or chronic edema of the breast.

Goal audience:

The patient group is defined as women who have undergone breast surgery, e.g. by removing part of the breast (as in breast-conserving surgery or mastectomy), breast reconstruction, augmentation or reduction. Patients with lymphedema under the armpit: People who suffer from lymphedema in the armpit area as a result of surgery, such as in breast cancer treatment, may benefit from wearing a compression bra to reduce symptoms. T

the purpose of the compression bra is to apply pressure to the breast area and armpit, which can help reduce swelling, support the breast(s) during the healing process, promote blood flow and reduce discomfort after surgery. Part of the

Part of the body for which the product is used:

Both the left and the right chest.

Practical tips

- Skin must always be correctly hygienic and dry.
- Remove rings, watches, bracelets and / or other jewelry when putting on the Caroskin™ Compression Bra.
- Cut long nails short.
- Take your time to put on the Vest correctly.
- Do not cut loose hanging threads.
- For possible repairs, contact your prescriber.

Wash & Care Advice

- Washing at 30 °C of the Caroskin™ Compression Bra with Ph neutral soap extends the lifetime. Through the Washing the clothing removes perspiration, skin fats and other substances that have influence on the elastic qualities of the vest.
- Frequent rinsing
- Do not wring, do not dry at heat source: allow air to dry

Warning

- Correct measurement:
 - Responsible: breast consultants
- Apply BH correctly:
 - Responsible breast consultants and nurses
- Maintenance of the bra:
 - see wash & care advice. This bra is for individual use and is standard or custom made. The use is only max 15 minutes per irradiation session, for maximum 15 days.

How do you determine the bra size?

The bra size consists of a number and a letter. The number (75-110) stands for the chest circumference (size or band size). The letter (A-E) stands for the cup size.

Chest circumference (size or band size = N2 size)

The chest circumference or band size is measured just below the bust. Take a tape measure and place the meter horizontally (front and back at the same height) around the chest just below the breasts. Exhale for a moment and let the meter fit tightly around your body, without pinching. The number of centimeters determines your circumference, rounded to 5 cm.

Chest size	75/80	85/90	95/100	105/110
Measurement N2	73 - 82 cm	83 - 92 cm	93 - 102 cm	103 - 112 cm

With the smallest breast circumference the hooks are closed on the 2nd row of eyes, with the largest breast circumference on the 1st row and this on both sides.

The Cup size

Important! this size is measured with a Bra on!

The cup size is determined by measuring the length of the breast (AB size). Take a measuring tape and place it at the start of the curve of the breast over the chest to the bottom of the breast border. The number of centimeters can be found in the table, which then correspond to a letter indicating the cup size.

Chest size	75/80	85/90	95/100	105/110
A	14 - 15 cm	15 - 16 cm	16 - 18 cm	-
B	15 - 17 cm	17 - 19 cm	18 - 21 cm	19 - 22 cm
C	17 - 19 cm	19 - 21 cm	21 - 23 cm	22 - 24 cm
D	19 - 21 cm	21 - 24 cm	23 - 25 cm	23 - 26 cm
E	24 +	25 +	26 +	-

* If the measured centimeters can't be found in the tables, we opt for made to measure.

- Machinewas / Lavage à la machine / Machine wash / Maschinenwäsche 30°C
- Niet droogzieren / Ne pas sécher en machine / Do not tumble dry / Nicht im Trockner
- Niet strijken / Ne pas repasser / Do not iron / Nicht bügeln
- Niet bleken / Ne pas javelliser / Do not bleach / Nicht bleichen
- Geen droogkuis / Ne pas nettoyer à sec / Do not dry clean / Nicht chemisch reinigen

Made by/Manufactured by:
Caroskin nv
's Gravenstraat 197 / E28
B-9810 Nazareth
www.caroskin.com

MD Medical Device