A complete scar care concept

silicone - drukkledij - hydratatie

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Caroskin nv
's Gravenstraat 197 / E28
9810 Nazareth
Belgium

Indications and action

- The *Caroskin™ Professional Silicone Sheets* are developed for the treatment and/or prevention of new and existing keloid and hypertrophic scars.
- The *Caroskin™ Professional Silicone Sheets* improve the elasticity of existing scars and reduce redness and raised scarring. The *Caroskin™ Professional Silicone Sheets* are also to be used as a buffer that prevents direct friction.

Description of the *Caroskin™ Professional gamma*

The *Caroskin™ Professional silicone sheets* are

- Self-adhesive silicone bandages, to be cut to size, for scar treatment; available in a *Light* (thin and elastic) version and in an *Elastic* (thick and elastic) version.
- The *Caroskin™ Professional silicone sheets* are made of 100% medical silicone gel, a **DOUBLE** elastic layer and a protective layer.

A range of *Caroskin™ Professional Handproducts* and special silicone bandages, to be cut to size, are used for post burn treatment, post (plastic) surgery and dermatology applications. The silicone gloves are made of 100% medical silicone gel and an elastic polyamide material.

Caroskin™ Cleaning Soap is an especially developed cleansing agent for the *Caroskin™ Professional products*.

Length of use

The *Caroskin™ Professional Silicone Sheets* for 2 to 8 weeks, depending on the site requiring treatment and conditions. The *Light* products lasts in general from 2-4 weeks, the *Elastic* version last in general for up to 8 weeks.

Directions for using and practical tips for the *Caroskin™ Professional Silicone Sheets*

- Cut the *Caroskin™ Professional Silicone Sheets* with rounded edges and cover the scar completely, with an overlap of approximately 2 cm. (fig.1-1)
- Remove the protective film from the sheet and stick the adhesive side onto the scar. (fig.1-2)
- Increase the use of *Caroskin™ Professional Silicone Sheets* with 1 hour per day up to a total of 20 hours. (fig.2)
- You can use tape or a bandage to make sure that the *Caroskin™ Professional Silicone Sheets* stay in place. (fig.1-5)
- When the adhesiveness is insufficient, clean the *Caroskin™ Professional silicone sheet* according to the care advice. Washing the product will increase its adhesiveness!

Wash & Care Advice for the *Caroskin™ Professional Silicone Sheets*

- Rub a few drops of an ecological friendly washing agent or *Caroskin™ Professional* into the *Caroskin™ Silicone Sheets*.
- The sheets lose their adhesive power after contact with the cleaning agent and/or water. Or wash the silicone sheet in the washing machine. (fig.1-7-8-9)
- Rub well to remove all dirt and rinse thoroughly under warm, running water.
- Place the (curled) *Caroskin™ Professional Silicone Sheets* on a dry tea-towel, sticky side up, to dry and let it uncurl. When dry, the adhesiveness will return. The better you wash and clean, the better the adhesiveness will be!
- Only use *Caroskin™ Professional Silicone Sheets* when they are fully dry.

Directions for using and practical tips for the *Caroskin™ Professional Handproducts (silicone glove and fingercap)*

- Respect the build up period/schema
- The silicone glove/fingercap is worn if there is scar formation on the hand or on the fingers. The action of the silicone glove makes the skin more supple and promotes the mobility of the joints.
- To pull the glove on, first fold back the edge of the glove.
- After pulling the glove on, turn the edge of the glove back into place.

Wash & Care Advice for *Caroskin™ Professional Handproducts (silicone glove and fingercap)*

- Turn the glove/fingercap inside out, rinse and rub in with an ecological washing agent or a few drops of the *Caroskin™ Cleaning Soap*. Or wash the silicone glove in the washing machine at 40°C.
- Rinse thoroughly with mild warm running water and let the glove/fingercap dry inside out.

Precautions:

- Always consult your doctor concerning skin irritation or skin problems. In many cases a more thorough hygiene of the product and a prolonged build up period of the use of the products will be helpful.
- *Caroskin™ Professional* products are for external use only.
- *Caroskin™ Professional* products are produced to exacting quality standards. If the *Caroskin™ Professional silicone sheet* should still lose silicone through wear-and-tear at the edges, you can trim the edges by sticking the silicone bandage on a piece of plastic foil and then trimming the edges.
- Never dry the *Caroskin™ Professional silicone sheet* in the sun or near a source of heat.
- *Caroskin™ Professional* products are entirely safe for treating children's scars. Nevertheless, keep *Caroskin™ Professional* products out of the reach of children.
- *Caroskin™ Professional* products are not sterile.

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